

## ■ CARE AND CLEANING



- When removing the appliance, grasp with both hands. To prevent breakage, do NOT warp or flex the appliance.
- Rinse appliance with warm water before insertion and after wearing. Store dry in supplied case for optimum protection.
- To clean, gently brush the appliance using a soft toothbrush and toothpaste.
- If necessary, soak the appliance in denture cleanser for up to 10 minutes daily. To avoid damage, do not soak longer than designated time.

# Clear-Lock

Retainers for Life™



*You've invested time and money in your smile. Protect that investment for years to come with Clear-Lock Retainers for Life™.*



## ■ WHAT IS CLEAR-LOCK?

---

Clear-Lock Retainers for Life™ provide a valuable service to patients who want to protect the esthetics of their smile. Through routine use and wear, retainers help maintain ideal tooth position so that the teeth are “locked” into place for years to come.

**CLEAR:** For the color of the biocompatible, BPA-free polymer from which the retainer is made

**LOCK:** Because it locks your teeth into place to maintain your best-looking smile

**FOR LIFE:** Your 3-D digital data is kept on file for 30 years, making a replacement retainer a quick and easy process — no impressions required!


## ■ HOW TO GET YOUR CLEAR-LOCK RETAINER

---

Your dentist may determine that you are an ideal candidate for Clear-Lock Retainers for Life. Once you agree to treatment, your dentist will take an impression of your upper and lower arch. This information is then sent to the dental lab and a custom model of your mouth is fabricated. The model is then scanned digitally and the 3-D file is sent to digital data storage, where it will be kept for 30 years.



Simultaneously, your model is used for the fabrication of Clear-Lock Retainers. In less than two weeks, the custom retainers will be delivered to your dentist's office for fitting. At that time, you will receive a set of three upper and/or three lower Clear-Lock Retainers, so that you always have spares on hand.

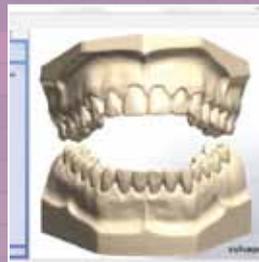

The 3-D digital data is stored for a period of 30 years. Should you lose or damage your retainers within this 30-year period, your dentist can prescribe a new set of Clear-Lock Retainers using your digitally-stored data. This frees you from returning to your dentist's office for an impression, saving valuable time!